

# FREE Pad-Pak™ Program

## Tell Us About an SCA

If you have used a HeartSine® samaritan® PAD during a Sudden Cardiac Arrest (SCA) in a public setting, it's critical that you TELL US, by providing HeartSine with the Saver EVO™ file that contains comprehensive event data about the event and a completed Customer Event Form. This information is used by HeartSine to measure the success of our Automated External Defibrillators (AEDs) and to continually improve our products to ensure optimum results. When you provide the Saver EVO file, along with a completed Customer Event Form, we'll send you a free Pad-Pak for your efforts!




### FREE Pad-Pak Process

- ▶ Immediately after the use of the HeartSine AED during an SCA, contact HeartSine to report the event and outcome.
- ▶ Immediately after the SCA event, follow the instructions in the Saver EVO manual to download the event data (.evo file) and submit it to HeartSine at [www.heartsine.com/defib](http://www.heartsine.com/defib). Please note that a pdf of the .evo file will not be accepted; only the original .evo file can be used for verification.
- ▶ Complete and submit at [www.heartsine.com/defib](http://www.heartsine.com/defib) the HeartSine Customer Event Form with information about the patient. Please note, only information requested on the Customer Event Form should be provided. All other patient information should remain anonymous.
- ▶ HeartSine will send the free Pad-Pak as soon as the Saver EVO (.evo) file and completed Customer Event Form have been received and verified. Please allow two business days for verification.

### Terms

- ▶ The event must be an actual sudden cardiac arrest to qualify.
- ▶ The event is verified by the HeartSine Clinical Team, whose decision is final.
- ▶ Exclusions apply. Please contact your sales representative for details.

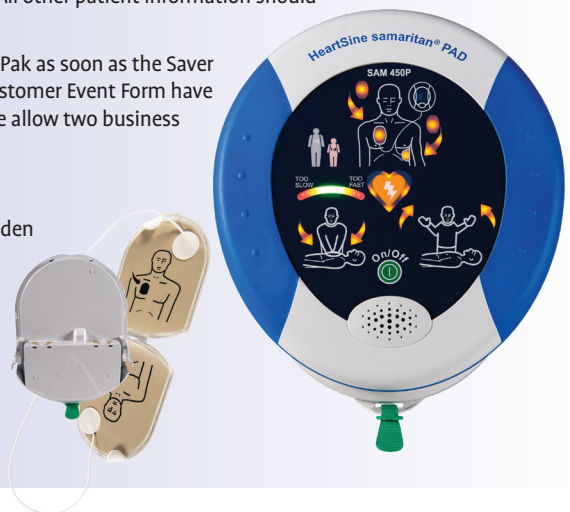


# HeartSine® samaritan® PAD Automated External Defibrillators (AEDs)

## BRIEF SUMMARY OF INDICATIONS AND IMPORTANT SAFETY INFORMATION

**INDICATIONS FOR USE:** The HeartSine samaritan PAD SAM 350P (SAM 350P), HeartSine samaritan PAD SAM 360P (SAM 360P) and HeartSine samaritan PAD SAM 450P (SAM 450P) are indicated for use on victims of cardiac arrest who are exhibiting the following signs: unconscious, not breathing, without circulation (without a pulse). The devices are intended for use by personnel who have been trained in their operation. Users should have received training in basic life support/AED, advanced life support or a physician-authorized emergency medical response training program. The devices are indicated for use on patients greater than 8 years old or over 55 lbs (25 kg) when used with the adult Pad-Pak™ (Pad-Pak-01 or Pad-Pak-07). They are indicated for use on children between 1 and 8 years of age or up to 55 lbs (25 kg) when used with the Pediatric-Pak™ (Pad-Pak-02).

**CONTRAINDICATION:** If the patient is responsive or conscious, do not use the HeartSine samaritan PAD to provide treatment.

**WARNINGS: AEDs:** The HeartSine samaritan PAD delivers therapeutic electrical shocks that can cause serious harm to either users or bystanders. Take care to ensure that no one touches the patient when a shock is to be delivered. • Touching the patient during the analysis phase of treatment can cause interference with the diagnostic process. Avoid contact with the patient while the HeartSine samaritan PAD is analyzing the patient. The device will instruct you when it is safe to touch the patient. • Do not delay treatment trying to find out the patient's exact age and weight. If a Pediatric-Pak or an alternative suitable defibrillator is not available, you may use an adult Pad-Pak. • The SAM 360P is a fully automatic defibrillator. When required, it will deliver a shock to the patient WITHOUT user intervention. • The SAM 450P CPR Rate Advisor is currently only intended to provide feedback on adult patients. If you treat a pediatric patient with the SAM 450P and an adult Pad-Pak, ignore any voice prompts regarding the rate of CPR. • Do NOT use the HeartSine samaritan PAD in the vicinity of explosive gases, including flammable anesthetics or concentrated oxygen. • Do NOT open or repair the device under any circumstances as there could be danger of electric shock. If damage is suspected, immediately replace the HeartSine samaritan PAD. **Pad-Paks:** • Do not use if the gel is dry. • The Pediatric Pad-Pak is not for use on patients under 1 year old. For use with children up to the age of 8 years or up to 55 lbs (25 kg). DO NOT DELAY THERAPY IF YOU ARE NOT SURE OF EXACT AGE OR WEIGHT. • Only HeartSine samaritan PADs with the  label are suitable for use with the Pediatric-Pak. If the HeartSine samaritan PAD you are using does not have this label, use the adult Pad-Pak if no alternatives are available. • The use of the Pediatric-Pak will enable delivery of 50J shocks to the pediatric patient. • The Pediatric-Pak contains a magnetic component (surface strength 6500 gauss). Avoid storage next to magnetically sensitive storage media. It is advised that Pediatric-Paks are stored separately when not in use. • Never charge, short circuit, puncture, deform, incinerate, heat above 85°C or expose contents of TSO (Aviation) Pad-Pak to water. Remove when discharged.

**PRECAUTIONS: AEDs:** • Proper placement of the HeartSine samaritan PAD electrode pads is critical. Electrode pads must be at least 1 in (2.5 cm) apart and should never touch one another. • Do not use electrode pads if pouch is not sealed. • Check the device periodically in accordance with the service and maintenance instructions provided in the User Manual. • Operate the HeartSine samaritan PAD at least 6 feet (2 meters) away from all radio frequency devices or switch off any equipment causing interference. • Use of the device outside the operating and storage ranges specified in the User Manual may cause the device to malfunction or reduce the shelf life of the Pad-Pak. • Do not immerse any part of the HeartSine samaritan PAD in water or any type of fluid. • Do not turn on the device unnecessarily as this may reduce the standby life of the device. • Do not use any unauthorized accessories with the device as the HeartSine samaritan PAD may malfunction if non-approved accessories are used. • Dispose of the device in accordance with national or local regulations. • Check with the relevant local government health department for information about any requirements associated with ownership and use of a defibrillator in the region where it is to be used. **Pad-Paks:** • Check expiration date. **Saver EVO™ Software:** • Download the complete HeartSine samaritan PAD memory prior to erasing it. This information should be stored safely for future reference. Ensure that only the events you want to delete have been selected prior to deleting. Once deleted from your computer's memory, events cannot be regenerated and all information will be lost.

**POTENTIAL ADVERSE EFFECTS:** The potential adverse effects (e.g., complications) associated with the use of an automated external defibrillator include, but are not limited to, the following: • Failure to identify shockable arrhythmia. • Failure to deliver a defibrillation shock in the presence of VF or pulseless VT, which may result in death or permanent injury. • Inappropriate energy which could cause failed defibrillation or post-shock dysfunction. • Myocardial damage. • Fire hazard in the presence of high oxygen concentration or flammable anesthetic agents. • Incorrectly shocking a pulse-sustaining rhythm and inducing VF or cardiac arrest. • Bystander shock from patient contact during defibrillation shock. • Interaction with pacemakers. • Skin burns around the electrode placement area. • Allergic dermatitis due to sensitivity to materials used in electrode construction. • Minor skin rash.


**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician.

Please consult the User Manual at [www.heartsine.com](http://www.heartsine.com) for the complete list of indications, contraindications, warnings, precautions, potential adverse events, safety and effectiveness data, instructions for use and other important information.



For further information contact us at [heartsinesupport@stryker.com](mailto:heartsinesupport@stryker.com) or visit our website at [heartsine.com](http://heartsine.com).

### EMEA/APAC

HeartSine Technologies, Ltd.  
203 Airport Road West  
Belfast, BT3 9ED  
United Kingdom  
Tel: +44 28 9093 9400  
Fax: +44 28 9093 9401 

### U.S./Americas

HeartSine Technologies LLC  
121 Friends Lane, Suite 400  
Newtown, PA 18940  
Toll Free: (866) 478 7463  
Tel: +1 215 860 8100  
Fax: +1 215 860 8192



UL Classified. See complete marking on product.

© 2019 HeartSine Technologies. All rights reserved.

H009-032-023-0 US